SUNSTAR

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**Ingredientes Não Medicinais - Goma de Mascar Fina:** Glycerin, diatomaceous earth (uncalcined), sodium silicate, water, flavor, titanium dioxide, monosodium phosphate, D&C Red #28, agar, methyl salicylate

**Ingredientes Não Medicinais - Cereja Fina e Hortelã Fina:** Glycerin, diatomaceous earth (uncalcined), sodium silicate, water, flavor, titanium dioxide, monosodium phosphate, D&C Red #33, FD&C Red #40, FD&C Blue #1, D&C Yellow #10, agar, sodium saccharin

**Ingredientes Nao Medicinais - Grânulos Médios - Chiclete:** Glycerin, pumice, sodium silicate, silica, water, flavor, titanium dioxide, monosodium phosphate, D&C Red #28, calcium disodium EDTA dihydrate, agar, sodium saccharin, methyl salicylate

**Ingredientes Nao Medicinais - Grânulos Médios - Menta:** Glycerin, pumice, sodium silicate, silica, water, flavor, titanium dioxide, monosodium phosphate, FD&C Blue #1, D&C Yellow #10, calcium disodium EDTA dihydrate, agar, sodium saccharin

**Ingredientes Não Medicinais - Cereja Média:** Glycerin, pumice, sodium silicate, silica, water, flavor, titanium dioxide, monosodium phosphate, D&C Red #28, agar, sodium saccharin, D&C Red #22

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SUNSTAR

**BUTLER**  **MONSTERZ****Prophy Paste**

Contains: 1.23% w/w Fluoride Ion from Sodium Fluoride

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Please refer to <http://us-professional.gumbrand.com/MSDS> for SDS information

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**BUTLER**  **MONSTERZ**

**Prophy Paste** with Fluoride

Butler® Monsterz Prophy Paste is specifically designed for polishing procedures as administered by licensed dental professionals. It consists of a unique mixture of polishing and cleaning agents and is supplied in the following grits:

- Fine:** Minimal stain and polishing needs
- Medium:** Normal polishing and cleaning indicated

Unit Dose Cup labels are color coded according to grit as follows:

- Fine:** Orange
- Medium:** Blue

**INDICATIONS:**

For cleaning and polishing procedures as part of a professional prophylaxis procedure. NOTE: PROPHYLAXIS PASTES CONTAINING FLUORIDE ARE NOT CONSIDERED A REPLACEMENT FOR OTHER FORMS OF FLUORIDE THERAPY.

**ADMINISTRATION:**

Butler® Monsterz Prophy Paste is to be used only by individuals who are professionally trained to perform a dental prophylaxis.

**PRECAUTIONS:**

Excessive ingestion of fluoride can cause nausea. Paste containing fluoride should be expectorated and NOT SWALLOWED.

Butler® Monsterz Prophy Paste with fluoride contains fluoride at a concentration of 1.23% fluoride ion.

**CONTRAINDICATIONS:**

Butler® Monsterz Prophy Paste with fluoride should not be administered to individuals with known allergic hypersensitivity to fluoride.

**DIRECTIONS FOR USE:**

Each cup contains enough Butler® Monsterz Prophy Paste for a single prophylaxis procedure.

Using a disposable prophy angle such as the Butler® Monsterz Disposable Prophy Angle or a sterilized conventional metal prophy angle, fill the cup by pressing the face of the cup into the paste while running the angle at low speed. Maintain a sufficient amount of paste on the cup throughout the procedure to avoid overheating the tooth surface. Saliva evacuation is recommended to avoid excessive dilution of paste and swallowing of the paste.

**INGREDIENTS:**

**Medicinal (in Canada Only) Ingredient:** 1.23% w/w Fluoride Ion from Sodium Fluoride

**Non-medicinal Ingredients - Fine Bubble Gum:** Glycerin, diatomaceous earth (uncalcined), sodium silicate, water, flavor, titanium dioxide, monosodium phosphate, D&C Red #28, agar, methyl salicylate

**Non-medicinal Ingredients - Fine Cherry and Fine Mint:** Glycerin, diatomaceous earth (uncalcined), sodium silicate, water, flavor, titanium dioxide, monosodium phosphate, D&C Red #33, FD&C Red #40, FD&C Blue #1, D&C Yellow #10, agar, sodium saccharin

**Non-medicinal Ingredients – Medium Bubble Gum:** Glycerin, pumice, sodium silicate, silica, water, flavor, titanium dioxide, monosodium phosphate, D&C Red #28, calcium disodium EDTA dihydrate, agar, sodium saccharin, methyl salicylate

**Non-medicinal Ingredients – Medium Mint:** Glycerin, pumice, sodium silicate, silica, water, flavor, titanium dioxide, monosodium phosphate, FD&C Blue #1, D&C Yellow #10, calcium disodium EDTA dihydrate, agar, sodium saccharin

**Non-medicinal Ingredients - Medium Cherry:** Glycerin, pumice, sodium silicate, silica, water, flavor, titanium dioxide, monosodium phosphate, D&C Red #28, agar, sodium saccharin, D&C Red #22

FLAVOR	TEXTURE	
	FINE	MEDIUM
BUBBLE GUM	✓	✓
CHERRY	✓	✓
MINT	✓	✓

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© **Sunstar Americas, Inc.** Schaumburg, IL 60195 U.S.A.